eakin Release™ limfjerningspray

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REF 839023
50ml e

TG Eakin Limited
15 Ballystockart Road,
Co. Down, BT23 5QY,
Northern Ireland, U.K.
www.eakin.eu

eakin Release™ adhesive remover

About this product: eakin Release™ adhesive remover spray is a silicone based, alcohol free liquid which can be used to assist with the removal of adhesive medical appliances. eakin Release™ adhesive remover spray is designed to reduce skin related trauma which may result from frequent removal of adhesive medical appliances. eakin Release™ adhesive remover spray interrupts the adhesive bond to the skin, resulting in easy removal of adhesive medical appliances without inhibiting adhesion of subsequent devices.

Intended users: For ostomates, individuals managing fistulae or wounds, healthcare professionals and caregivers.

Instructions for use: Spray eakin Release™ adhesive remover spray around the edge of the adhesive medical appliances and wait a few seconds. Gently peel appliance away from the skin using additional eakin Release™ spray if necessary. eakin Release™ dries in seconds. Once dry, adhesion of subsequent appliances will not be affected.

Warnings and cautions: Pressurised container – protect from sunlight and do not expose to temperatures exceeding 50 degrees Celsius. Do not pierce or burn, even after use. Do not

spray on a naked flame or any incandescent material. Keep away from sources of ignition – no smoking. Keep out of reach of children.

Avoid use on delicate areas of the body (eyes, mouth, ears, nose, genitalia). If contact occurs, flush with water and seek medical advice. Do not inhale the spray and avoid spraying in locations where inhalation could occur. Do not use on premature infants. eakin Release™ adhesive remover spray is not appropriate for use in incubators. Consult a healthcare professional if you have any skin irritation, redness or discomfort in the area of product application prior to use. If during use you experience any skin irritation, redness or discomfort in the area of product application, discontinue use and consult a healthcare professional. Do not apply to broken skin or open wounds.

Other information: Store between 10 and 50 degrees Celsius. eakin® accepts no liability for any injury or other loss that may arise if this product is used in a manner contrary to its current recommendations. Any serious incident encountered in relation to the use of this device should be reported to the manufacturer and the competent authority in the country of use. A serious incident is one involving serious deterioration in health or patient death.

eakin Release™ adhesive remover spray

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Publicare AG