
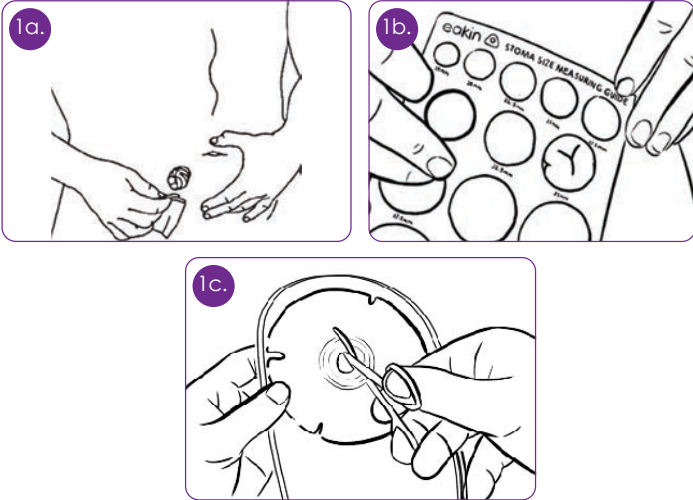
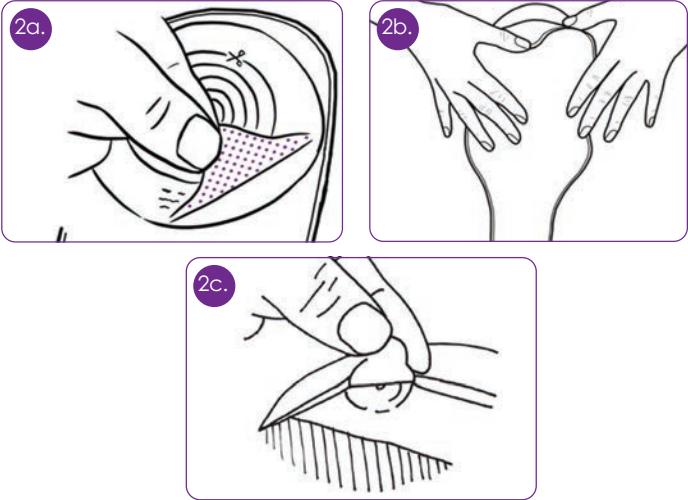
1.


2.

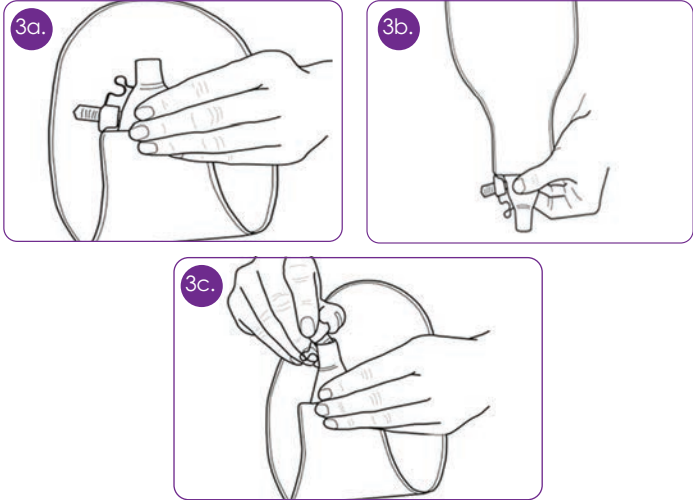3.

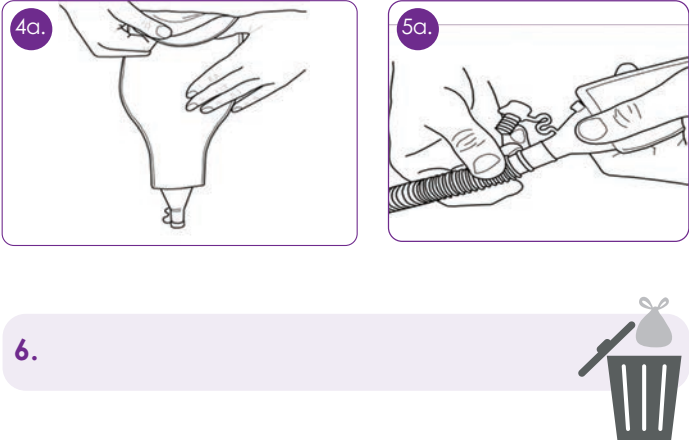4.

EN Instructions for use

1. Preparation: Ensure the area around the stoma is clean and dry **1a**. If the pouch is pre-cut, please proceed to **step 2**. If the pouch needs to be cut to size, use the enclosed measuring guide to measure the size and shape of the stoma **1b**. Using the cutting guide on the baseplate cut a hole to fit the exact pre-measured size and shape of the stoma using curved scissors, taking care not to cut through the pouch film on the other side **1c**. An accurate fit reduces the risk of leaks and skin problems.

2. Application: Peel the protective film off the baseplate **2a**. Align the centre hole on the baseplate with your stoma. Press gently against your skin, starting just below your stoma, smoothing upwards and outwards **2b**. A filter cover may be used when bathing or to prevent 'pancaking'. Apply directly over the filter, which is seen as a dark circle on the top of the pouch **2c**. (After bathing, remove filter cover).

3. Emptying: Lift the bung up, remove the stopper and fasten at the side **3a**. Point the bung down and empty into a toilet. To control the flow, squeeze the area above the bung **3b**. To close, re-insert the stopper into the bung **3c**.

4. Removal: Gently peel the baseplate away from the skin **4a**.

5. Connection to a drainage bag: Remove the stopper from the bung. Insert bung into the drainage tubing **5a**.

6. Disposal: Place used products into a waste bag and waste bin **6a**. Do not flush the product down the toilet **6b**. Dispose of waste products in accordance with local guidelines.

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